




Indicated for  
Prevention thru  
Stage III, IV,  
Flaps, Grafts


## True3 Digital EXTRA Mattress System – Bariatric

New Generation 3 Technology - TRI-PHASIC, Multiple Cell Alternating Technology. Created to give clinicians an alternative to immersion technology (low air loss or air fluidized) while improving patient outcomes by providing TRI-PHASIC Alternating True Pressure Relief. Change the dynamics of offloading pressure ulcers with a Support Surface that is “Active” instead of “Reactive”.

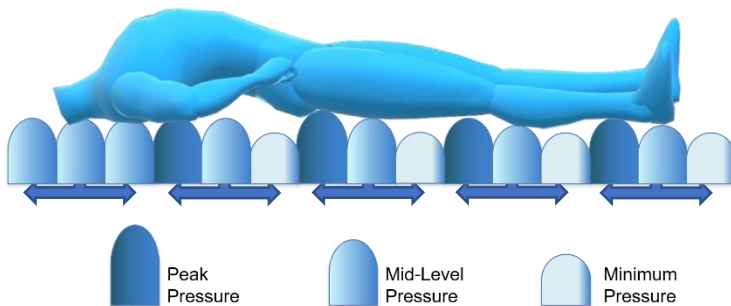

The True3 Digital Mattress System does not function by immersion for body mass distribution of pressure. Instead, the pressure alternates cyclically through a series of 3-cell groupings throughout the mattress. This TRI-PHASIC Alternation ensures pressure relief for one-third of the body every 2.5 minutes (full body pressure relief every 7.5 minutes).

TRI-PHASIC Technology uses a 3-Cell alternating cycle (A, B, and C) which automatically changes the pressure in the cells every 2.5 minutes.

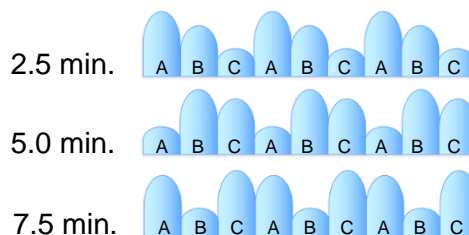

Cyclical pressure alternation helps prevent prolonged, damaging capillary occlusion and stimulates the flow of blood (reactive hyperemia) and lymphatic fluids to provide oxygen and remove waste.

TRI-PHASIC pressure alternation provides the Support and the Offloading that allows the patient to be placed “on the system” instead of “in the system”. This relieves the back, hip, and neck pain resulting from “immersion” and increases patient compliance.

### Features and Benefits

“Active” vs “Reactive” Technology  
True Offloading through Sequential Whole-Body Pressure Relief

TRI-PHASIC Alternation to promote Reactive Hyperemia

Self-Contained Design – No Noise or Heat from constant Blower Fan

Infection Control – No Airflow out of cells, across patient, and into room

Fail Safe “3rd Layer” of air cells to Support Patient in Power Failures

3-Cell Support for Comfort – Patient lies “on” not “in” the mattress for a more normal posture with less neck, back, and hip pain

No Special Linens or Expensive Breathable Incontinence Pad Use

Simple to Use – Strap Down, Attach Pump, Plug In & Turn On

Prominent CPR Release at handle attachment to Pump

FULL 2 YEAR WARRANTY against manufacturer defects

[www.true3digital.com](http://www.true3digital.com)

Phone: 1-866-968-6352 | Fax: 1-866-968-6353

WoundKair Concepts, Inc. | [www.woundkair.com](http://www.woundkair.com) | Email: [customerservice@woundkair.com](mailto:customerservice@woundkair.com)

# True3 Digital EXTRA Mattress System - Bariatric Specifications

## Comfort Level

– 8 available settings from soft to hard

## Alarm Mute

– Press to silence alarm

## Mode Select

– Press to toggle between Alternating/Static

## Panel Lock

– Press and Hold to Lock/Release Controls

## Max Inflation


– Press for Maximum Patient Support

## Power Button

– Press to Toggle Pump On/Off

## Simple Controls – Easy to Use – Automatic Function

### The True<sup>3</sup> Digital Pump

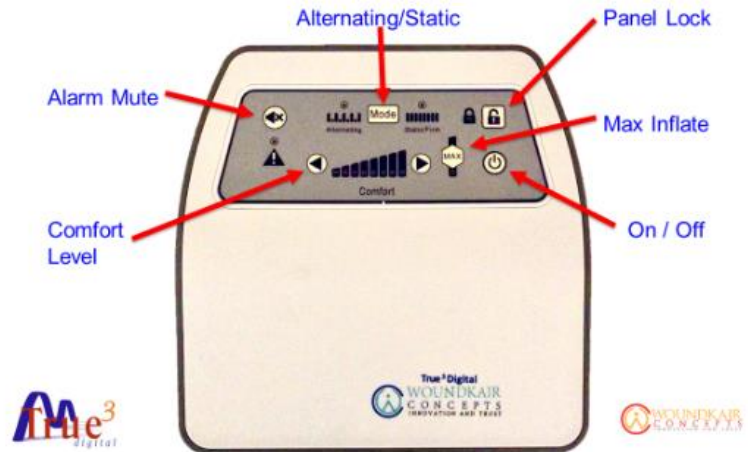

Item Number	Product Description		
TTDE-3030	True3 Digital EXTRA Mattress System (Bariatric - Full Mattress Replacement)		
COVTTD-B	True3 Digital EXTRA Replacement Mattress Cover (Bariatric)		
PUMPTTD-B	True3 Digital EXTRA Replacement Pump (Bariatric)		
MATTTTD-B	True3 Digital EXTRA Replacement Mattress (Bariatric)		
EXTWTTD	True3 Digital Mattress System Extended Warranty Program (24 month)		
Mattress Specifications		Pump Specifications	
		Power Supply	110V / 60HZ
Maximum Weight	750 lbs.	Air Output	>10 liters per minute
Dimensions	80" x 48" x 8"	Dimensions	10.16 (L) x 3.35 (W) x 10.16 (H)
		Weight	6.0 lbs.

[www.true3digital.com](http://www.true3digital.com)

Phone: 1-866-968-6352 | Fax: 1-866-968-6353

WoundKair Concepts, Inc. | [www.woundkair.com](http://www.woundkair.com) | Email: [customerservice@woundkair.com](mailto:customerservice@woundkair.com)