



TrueKAST with Quicksaw Technology Total Contact Casting System

The *TrueKast* with patent pending *Quicksaw Technology* has a clinician-driven design providing one of the lightest, strongest, and safest TCC available today. Quicksaw Technology allows for the safe and fast removal of the TCC with NO CAST SAW NEEDED. With an application time of less than eight minutes and a weight-bearing dry time of 8 minutes, the TrueKast provides the “Gold Standard” in offloading diabetic foot ulcers.



EVERYTHING YOU NEED IN ONE KIT

- ✓ Light, Traditional Rigid Cast Wall Support of Patient’s Weight Providing a True TCC “Gold Standard” of Offloading
- ✓ Attached Walking Surface Options with Advanced “Lock and Walk Technology” SFP, LFP & TMA. XL & XXL Cast Shoe option for unstable patients (Options may be purchased separately as well)
- ✓ TFP Foam System to Protect the Foot, Achilles, and Malleolus
- ✓ Fast & Safe Quicksaw System - NO CAST SAW NEEDED



Quicksaw Technology

Item No.	Description	Kits / Case
TKQS-SFP	TrueKast with Quicksaw, Small Foot Plate	5/case
TKQS-LFP	TrueKast with Quicksaw, Large Foot Plate	5/case
TKQS-TMA	TrueKast with Quicksaw, TMA Foot Plate	5/case
TKQS-XLCS	TrueKast with Quicksaw, XL Cast Shoe	5/case
TKQS-XXLCS	TrueKast with Quicksaw, XXL Cast Shoe	5/case

WoundKair Concepts, Inc.

Phone: 1-866-968-6352 | Fax: 1-866-968-6353

www.truekast.com | www.woundkair.com | Email: customerservice@woundkair.com

WoundKair Concepts' TrueKast w/Quicksaw

Finally, the Traditional Total Contact Cast has received a 21st Century update. We started with the best of the traditional TCC features that have made it the “Gold Standard” for offloading diabetic foot ulcers: A Rigid Cast Wall combined with an Embedded Posterior Splint, Enforced Compliance with an Attached Walking Surface, and a Wrapping Pattern that ensures Intimate Contact and Conformity to Effectively Transfer Weight from the Plantar surface to the Calf. Then we added New and Improved Technologies such as: The Quicksaw Removal System, Lock and Walk Footplates, Fast Drying Cast Tape, and the TFP foam protection system. With Quicksaw Technology, Woundkair Concepts has produced the first commercially available TCC that DOES NOT NEED A CAST SAW for removal.

Indications and Contraindications of a Total Contact Cast

INDICATIONS

- Diabetic foot ulcers on heel, mid-foot, or plantar
- Non-infected neuropathic foot ulcers
- Charcot Neuroarthropathy
- Post-reconstruction or post-operative care
- Delayed primary closure

CONTRAINDICATIONS

- Infection
 - Ulcer deeper than width
 - Excessive foot or leg swelling
 - Poor vascular status
 - Non-compliant with visits, etc...
-

Patient Information

Please have your patient follow these recommended guidelines to insure they receive the maximum benefit of the TrueKast Total Contact Cast with QuickSaw.

YOU SHOULD

- Elevate your foot above your heart level if you are experiencing swelling.
- Reduce your ambulation and be cautious of strenuous activities that could cause injury.
- Be prepared to have someone drive you if your cast is placed on your right foot.
- Move toes frequently.

YOU SHOULD NOT

- Do Not remove the blue cohesive bandage.
- Do Not insert any item inside the cast.
- Do Not insert any powders or creams inside the cast.
- Do Not get your cast wet.
- Do Not break off any portion of the cast.
- Do Not open the plastic sheath which contains the Quicksaw.

CONTACT YOUR PHYSICIAN IF ANY OF THESE OCCUR

- A feeling of pressure, discomfort, numbness, or constant tingling.
 - A break or soft spot develops in your cast.
 - A severely soiled cast or a loose-fitting cast.
 - Pain, swelling or fever after attempting elevation, pain medication, or cold compress.
-

**To place an order or to request a demonstration please call
1.866.968.6352**

